


We care more



Freedom
through
combination

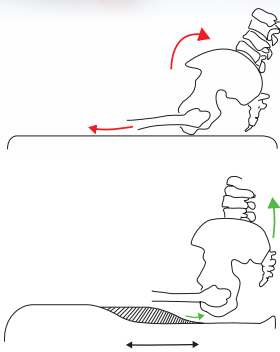


Integrated seat pocket

An integrated seat pocket helps facilitate activity, postural control, and reduces shear. Other systems using a flat surface leads inevitably to posterior tilt of the user's pelvis.


Step 1 adjust the seat depth to suit upper leg length

Step 2 adjust the seat pocket for perfect pelvic support

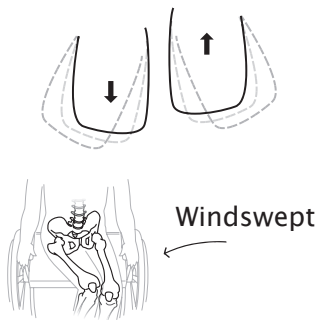


Adjustable leg parts

Seat depth, adduction, and abduction are all seamlessly and precisely adjusted. The system enables you to adjust seat depth up to 10 cm for the most precise fit imaginable as well as enabling your solution to "grow" with the user.

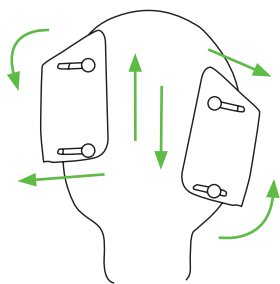

Adjustable back

The three-way lateral support aids in improving postural control, lessening deformity, thus making it possible to achieve a higher level of activity. Here you can either correct or compensate deformities, making it possible to maintain an optimal seated position.

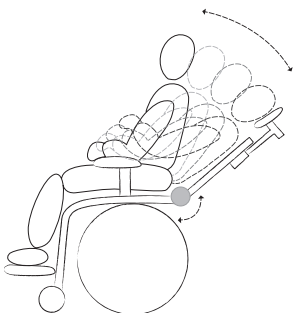

Flex-joint

Rigid supports are needed to create a functional, seated position.

Dynamic supports offer controlled movement, variation, and dampen the effects of extension spasticity. Zitzi Pengy comes equipped with a flex-joint to counteract spasticity and offer a dynamic sitting.

Pengy seat unit

Sun cover

Extendable sun cover protects the child from weather and wind.

Coolmax®

Cushion with Coolmax fabric for optimal comfort.

Flex-joint

Reduces the effects of extension spasticity.

Accessories included:

Arm rest
Hip belt
Coolmax cushion
Upper support bracket

Adjustable back

Improving postural control.


Abduction/adduction/ windswept

Easily adjusted to fit the users needs.



PENGY SEAT UNIT

SIZES.....	1
USER WEIGHT.....	45 kg
COLOUR.....	Black, orange, blue
WEIGHT.....	7 kg
CRASH TESTED.....	Yes, approved
SEAT DEPTH.....	28-38 cm
CHEST WIDTH.....	19-28 cm
HIP ANGLE.....	-10/+30
AXILLA HEIGHT.....	20-35 cm

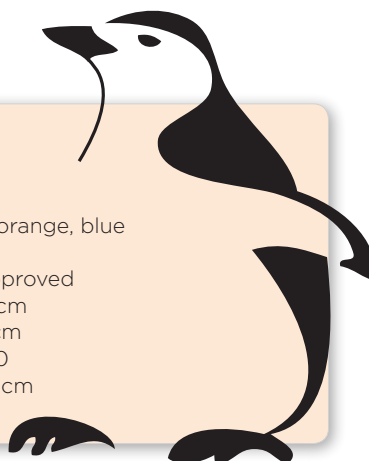

PENGY SULKY

SIZES.....	1
WHEELS.....	7", 12", terrain wheels
LOAD WIGHT.....	50 kg
TILT.....	-23° to +40°
SEAT HEIGHT.....	54 cm
COLOUR.....	Black
WEIGHT.....	12 kg
CRASH TESTED.....	Approved for 43 kg



Finally a stroller with well positioned seating!

Integral Dynamic Suspension

Zitzi Pengy has a built-in dynamic suspension system that allows it to roll smoothly on uneven surfaces.

Simple folding

Zitzi Pengy folds through a simple manipulation system and stays in its folded position automatically.

Removable and reversible seat unit

Crash tested

Crash tested and approved

Terrain wheels (accessory)

Can be equipped with terrain wheels which is great during the winter or for walks in the forest.

Advanced Integrated Seat System

Zitzi Pengy chair is the sister chair to our Delfi Pro and has been specially developed to fit Zitzi Pengy buggy. Zitzi Pengy has the same features as Delfi Pro. Features include adjustable seatpocket, flex function, adjustable lateral support along with a host of other features.

Quick release wheels

Direction lock



Freedom through combination.



Pengy seat



Delfi Pro



Anatomic Sits & Soft Sits



Carseat Pro



Terrain wheels
(accessory)

Accessories

Pengy seat



Black/black cover



Black/blue cover



Black/orange cover



Sun cover incl bracket



Table



Head rest size 1 black



Head rest size 1 long black



Head rest size 2 black



Head rest size 2 long black



ProSerie Plush



ProSerie single lower



ProSerie Junior



ProSerie Adult



ProSerie Mini



Bracelet



Sternum support Pro



H-strap Pro



Anti rotation belt



Hip belt



Ankle holder

Accessories

Pengy sulky



Footrest



Calf support




Basket



Transportation kit




Mounting system CarseatPro

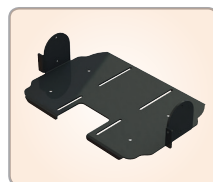


Mounting system Delfi Pro



Terrain wheels

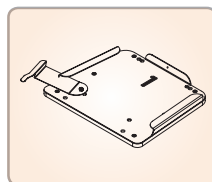

Mounting plate reinforced size 1/2/3

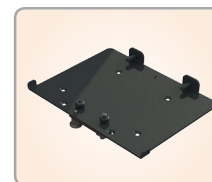

Mounting plate size 1/2/3




Adapter System Universal

Technical data

Pengy seat



FOOTREST FOLDABLE

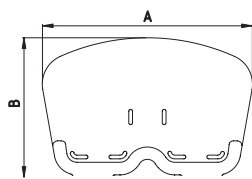




A x B
Size 1 315 x 195 mm

SEAT UNIT

Windswept

Abduction/Adduction

Seat depth/
Leg length

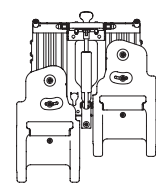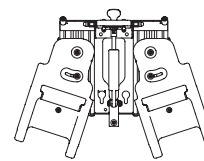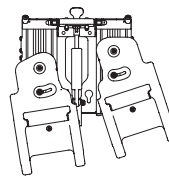


Seat depth adjustment:
28-38 cm

Abduction:
0° till +18°

Windswept:
Windswept possibility 9°


Adduction:
0° till -9°

ARMREST

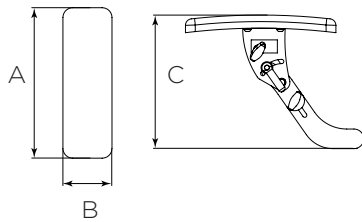

C
Size 1 140-205 mm


HEADREST

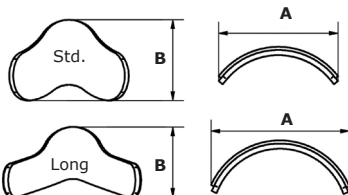

	A x B
Head rest size 1	220 x 150 mm
Head rest size 1 long	280 x 160 mm
Head rest size 2	240 x 190 mm
Head rest size 2 long	280 x 210 mm

ADJUSTABLE BACK

Measurements inc. cover

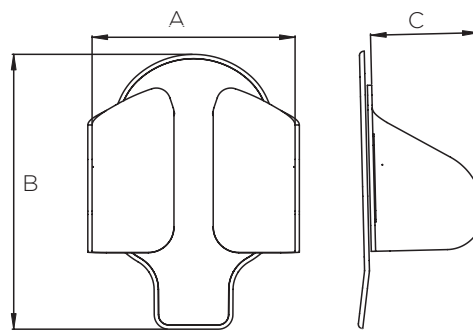

A - Chest width adjustment
190 - 280 mm

B - Back height adjustment
350 - 430 mm

C - Chest depth
150 mm

SIZES.....1
 USER WEIGHT.....45 kg
 COLOUR.....Black, orange, blue
 WEIGHT.....7 kg
 CRASH TESTED.....Yes, approved
 SEAT DEPTH.....28-38 cm
 CHEST WIDTH.....19-28 cm
 HIP ANGLE.....-10/+30
 AXILLA HEIGHT.....20-35 cm

Technical data

Pengy sulky

