



APPLICATION

\*LOW/MEDIUM RISK USER

Sizes: Up to 18" x 18"  
Larger sizes available POA.

Community One

C1



www.mmsmedical.ie

MMS Medical  
51 East Gate Drive,  
Little Island,  
Cork.  
T: 021 461 8000  
F: 021 461 8099  
E: sales@mmsmedical.ie

MMS Medical  
Unit 81,  
Western Parkway Business Park,  
Lower Ballymount Road,  
Dublin 12.  
T: 01 450 0582  
F: 01 456 5889  
E: sales@mmsmedical.ie



# Community One

<b>Description:</b>	<p>The Community One is a special bi-directional cushion constructed in multi-layer format, consisting of both reflex and memory foams. This results in an extremely comfortable and high-performance product with the added bonus of making installation very user friendly.</p> <p>This product fulfils the design directive for the Community range, ensuring high levels of comfort, performance, durability, and safety.</p>
<b>Sizes:</b>	Up to 18" x 18". All cushions are 3" in thickness. Larger sizes available POA.
<b>Cover:</b>	Vapour permeable cover with a concealed heavy duty zip and internal vapour permeable thermo plastic liner.
<b>Risk Category*:</b>	Low/medium
<b>Weight Guide†:</b>	For maximum comfort, durability and performance the weight guide on a 17" x 17" is 14 - 15 stone (89-95kg)
<b>Support Pressure:</b>	1lb per square inch at 50% compression or deflection
<b>Linear Load Limit:</b>	21 stone (133 kg) on a typical 17"x17" cushion
<b>Availability:</b>	Individually or in bulk (bulk prices by negotiation)

\* Risk categories are given as a guide and rely on your judgement and experience.

† Weight Guides are a general indication of an ideal user weight to give optimum durability, comfort and performance; it is not a definitive or maximum value. (This product has been tested to 325lb (146kg) without failure).

## Instructions for Use and Cleaning

### DESIGN

The device has been designed to provide the correct levels of pressure reduction, support and comfort which are needed and according to the design the correct amount of postural assistance.

### PLACEMENT

1. Before using the cushion for the first time, please remove the outer packaging if present and check that the chair seat has no objects or projections which may damage the new cushion.
2. Place the device onto the chair ensuring that our label is on the base of the device (ie. do not sit or lean on the label side).

### CLEANING

The cushion has removable covers and can be cleaned according to the material type. If possible sponge off any spillage or contamination as soon as possible with hot water and soap. Do not remove the special transparent liner (if fitted) but sponge it down.

**DO NOT PLACE THE WHOLE DEVICE DIRECTLY INTO A WASHING MACHINE**

### DOMESTIC CLEANING

All products have an approved cleaning label which you should follow to ensure durability and cleanliness of the product.

ALL WATERPROOF COVERS wipe over with hot water and soap, a mild hypochlorite solution can also be used (Milton) if desired. Vapour permeable material, towelling and spacer covers can be machine washed at 60 degrees and tumbled dry on warm only. Leave to air dry before refitting. Do not use furniture polish on the cover.

### CLEANING INSTITUTIONAL VAPOUR PERMEABLE MATERIAL

Can be washed at 60 degrees Celsius or dry cleaned. Wipe over with full strength Sodium Hypochlorite or Sodium Dichloroisocyanurate and maintain surface contact for 2-5 minutes. Wipe off and rinse with clean water and dry.

### DO NOT USE PHENOL BASED PRODUCTS ON ANY WATERPROOF COVER

If MRSA is suspected or confirmed please contact your Infection Control Department for advice.