

Features of the Serena II wheelchair



1	Headrest	7	Foot plate	13	Armrest
2	Seat cushion	8	Steering wheels	14	Side panel
3	Legrest	9	Seat & back adjustment	15	Drive wheels
4	Hand brake	10	Push handle	16	Quick-release axle
5	Calf pad	11	Assistant brake	17	Tip assist
6	Heel strap	12	Back cushion	18	Anti-tip wheel

Preparing the operating condition

Generally, your wheelchair is delivered completely assembled, folded. To make it ready for use, just a few easy and simple handgrips are required.




Danger of getting trapped!

Watch your fingers when folding and setting up the wheelchair and only grasp the parts illustrated.

Backrest

The backrest can be folded down to transport the wheelchair in a space saving manner.

To restore the function of the backrest, fold the backrest upwards again and fasten it to the gas spring with the connecting pin ①.

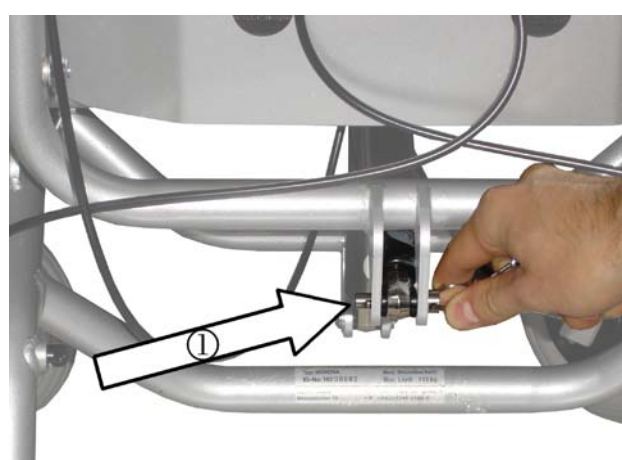

Back cushion

The back cushion is secured to the backplate by means of a Velcro tape.



Seat cushion

The seat cushion is secured to the seatplate using a Velcro tape. Make sure the seatplate ends flush with the backplate.




Side panels

The side panels can be fixed with just a few handgrips.

To do this, move the side panels into the provided mounts.

Tighten the lever screw ① to attach the side panels.

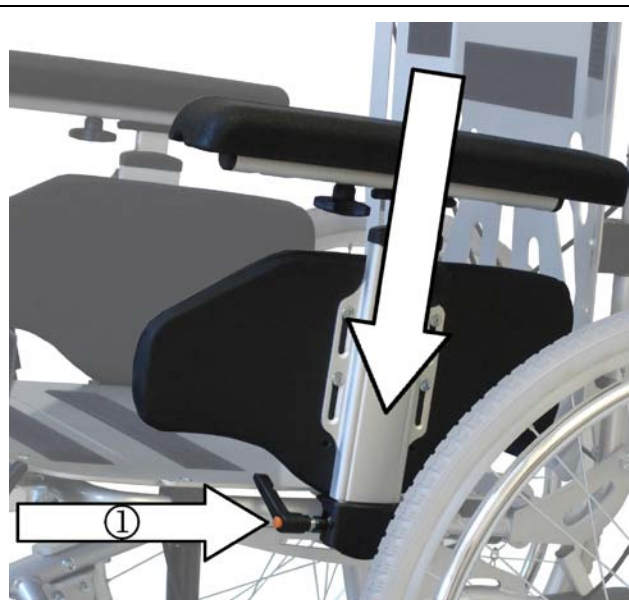


Legrests

Mount the legrests in the front.

Make sure the legrests are firmly locked in position.

Fasten the legrests with the lever screw.

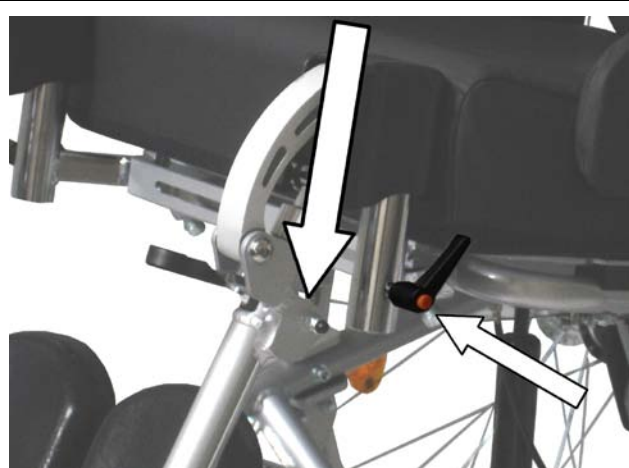

Calf pads

The calf pads can be mounted with just a few handgrips.


Insert the calf pad into the provided borehole on the legrest.

To attach the calf pad, tighten the lever screw.



Foot plates

Tilt down the foot plates.

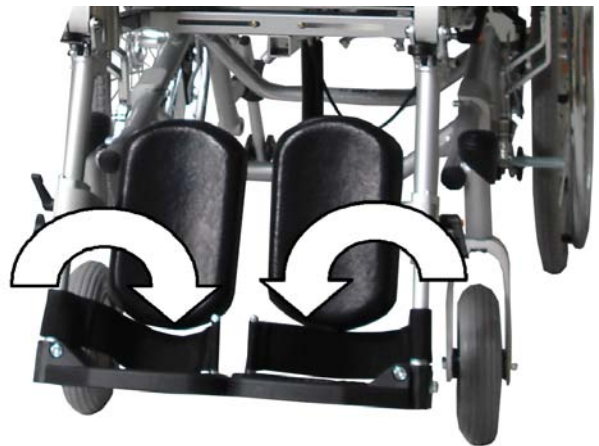

Headrest

The headrest can be mounted quickly and easily.

Insert the fastening rod of the headrest into the opening on the backplate.

Tighten the lever screw to mount the headrest.



Initial settings


Headrest

Both angle and position of the headrest can be adapted to the user.

Height of the headrest

Loosen the butterfly screw with which the headrest is fastened.

Move the headrest into the desired height position, then tighten the screw again.

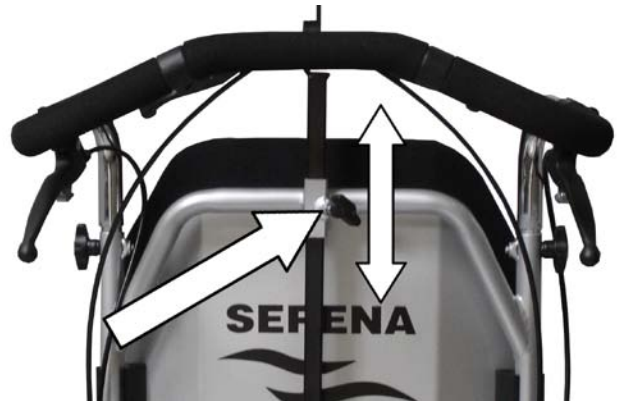

Depth of the headrest



Loosen both butterfly screws at the fastening of the headrest.

Move the headrest to the front or rear until the desired position is reached.

Tighten the butterfly screws screw again to lock the headrest in place.



Angle of the headrest

<p>You can tilt the headrest forwards.</p> <p>To do this, loosen one of the two butterfly screws① and swivel the headrest forwards or backwards.</p> <p>To lock the headrest in the desired position, tighten the screw again.</p>	
<p>In addition, the headrest can be pivoted to the left or the right.</p> <p>To do this, the butterfly screw② at the headrest must be loosened.</p> <p>When you have moved the headrest into the desired position, fasten it again by means of the butterfly screw.</p>	



MMS Medical
51 East Gate Drive,
Little Island,
Cork

Tel : 021 4618000
Fax : 021 4618099
Email: info@mmsmedical.ie
www.mmsmedical.ie