

Benefits

For the Fitter:

- easy to fit (headrest can be shaped by hand)
- attaches to standard three point hardware
- optional use of headband (included) for additional anterior support
- the pads are easily reshaped by hand when the occupant is repositioned

For the Wearer and/or Caregiver:

- a cool and comfortable headrest, the modular design of the cushions allows for air circulation
- easy to put on and remove
- low profile; unobtrusive appearance
- easy to keep clean (sponge clean with soap and water, then air dry)
- total control for the head
- low heat retention

Sizes and Colours



- Available in Adult, Small and Pediatric sizes
- Currently available in Black only

MMS
MEDICAL

Accessories

Side Pad

Part # SAVANT-SP

For extra lateral support. This support can also be used as a switch mount.



Whitmyer/Stealth Adaptor

Part # SAVANT-WA

This adaptor allows the Savant to be fitted to 19mm ball Whitmyer and 17 mm ball Stealth mounting hardware.



Netti Adaptor

Part # SAVANT-NA

This adaptor allows the Savant to be fitted to Netti mounting hardware.



R82 Adaptors

These adaptors allow R82 headrests and hardware to be fitted to the Axion, Savant Headrest and Twin Hardware.



Twin Headrest Hardware

Part #TWIN HH

This hardware allows the Axion 6025 and Axion 9025 to be fitted to a wheelchair back. The Axion 6019 and Axion 9019 can be fitted to Whitmyer and Stealth hardware.



More Fine Products from Symmetric Designs

Headmaster
Collar




TOT
Collar



Knee & Elbow
Orthoses



www.symmetric-designs.com CE

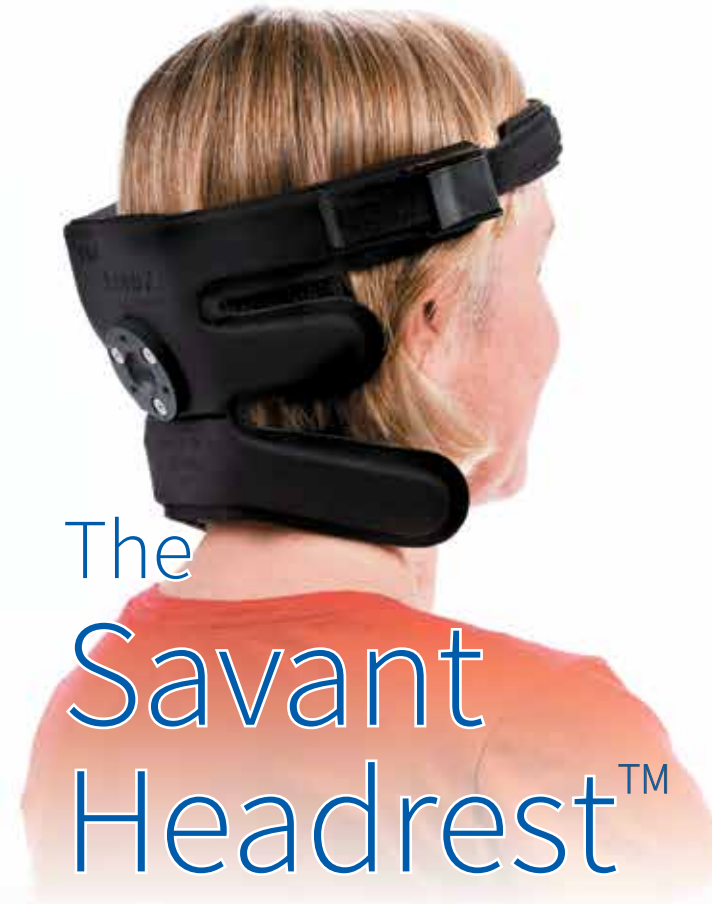

The Savant Headrest™

A Wheelchair Headrest for Total Head Control

The Savant Headrest™ is a new and unique support for the head. It is cool, light-weight and can easily be shaped by hand. It provides support at three levels on the head to control lateral flexion and rotation. A headband can be used for anterior support.

i symmetric
designs

Fitting Instructions

Choosing the Correct Size

The Savant is available in three sizes:

- Adult
- Small
- Pediatric



Sizing: On the Client

Circumference at forehead	Inches	cm
Adult	22 - 24	56 - 61
Small	18 - 22	46 - 56
Pediatric	14 - 18	36 - 46
1/2 Circumference behind ears		
Adult	8 - 9	20 - 30
Small	6 - 7	15 - 18
Pediatric	5 - 6	13 - 15

Sizing: On the Headrest

Inside distance along top support	Inches	cm
Adult	14	36
Small	12	30
Pediatric	9	23
Inside distance along middle support		
Adult	8	20
Small	7	18
Pediatric	5	13
Height of Headrest		
Adult	6	15
Small	5	13
Pediatric	4.5	12



The Savant is constructed with a unique steel core that allows selective shaping without tools.



The mounting disk accommodates a 1" (25mm) ball. The disk can be removed to attach custom hardware using M5 screws.

Netti, Whitmyer and Stealth hardware adapters are available.



The Savant comes with a headband that can be used for anterior support.

The Savant can be worn with or without the headband.

Features

The Savant Headrest™ is a unique new support for the head. It is cool, light-weight and can easily be shaped by hand. It provides support at three levels on the head to control lateral flexion and rotation.

A headband can be used for anterior support.

- Low profile headband gives anterior support
- Quick release clamps hold the Headband in place. Easy release for fine adjustment.



- High Lateral Pads give effective lateral flexion control
- Mid-Lateral Pads are shaped for comfort behind the ears
- Sub-Occipital Pads form extra support below the mandible.
- The pads are filled with a soft gel for comfort.
- The Savant is covered with a soft and durable nylon lycra knit fabric.

Symmetric Designs accepts no liability resulting from the misapplication of this product.