

beka hospitec



MMS
MEDICAL

AVERO VIVA

The classically elegant care bath

AVERO VIVA

offers safety and comfort ...

beka
hospitec



The classically elegant design for your care bath ...

The distinctiveness of the AVERO VIVA tub lies in its timeless design, reflected by the clean, simple outer panels and a very beautiful inside bathtub. The ergonomic inside of the bathtub offers residents sufficient space in the shoulder area and ensures that appropriate seats and bath lifters can be used. The AVERO VIVA lift bathtub can be fully accessed underneath, meaning that bath lifters can be used without any problems in the smallest of space. Armrests with high-quality chrome-plated brass handles, guides for the foot rests embedded in the edge of the bath and scald protection for the hand shower and bath tap provide maximum comfort and safety when bathing. The bath is operated using a keypad with symbols.



AVERO VIVA

impresses with its classically beautiful bathtub ...




Advantages for your carers and residents ...

- ❑ Simple operation using operating or wired remote control
- ❑ Homely look
- ❑ Option: change of viewing direction
- ❑ Option: legionella rinse
- ❑ Ergonomic work with bath height adjustment
- ❑ Pore-free surface for simple cleaning and optimum hygiene
- ❑ Armrests with handles for more comfort and safety
- ❑ Scald protection for hand shower and bath tap
- ❑ Lots of shoulder space for the resident
- ❑ Suitable for all standard lifts



AVERO VIVA options

Spa and therapy ...

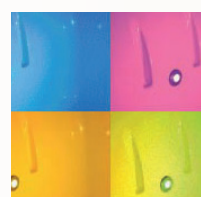

Coloured light therapy system ...

Two LED spotlights integrated in the sides of the bathtub can immerse the bath water in the full colour spectrum without heat radiation and disturbing glare effect for the resident. The coloured light therapy system has a choice of colours to select from as well as an automatic sequence for a colour change program.



Air-Jet Whirl System ...

The jacuzzi bath with its cleansing and reviving effect offers a pleasant massage effect and is particularly soothing for the resident. Due to a special process, contamination of the integrated nozzles is not possible meaning that the use of an integrated disinfection system to disinfect the jacuzzi bath is not necessary.




Music sound system ...

With two integrated sound pressure oscillators in the top section, the vibrations of the music are transferred directly into the water and onto the resident's body and allow individual music files to be played as required.



Aromatherapy system ...

An aromatherapy system integrated in the bathtub allows the use of different aromas, which, together with the warm air from the jacuzzi system and the bath water, create a wonderful sensual experience for the resident. This option is only possible together with the jacuzzi system.

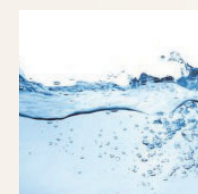

Automatic water stop ...

Enjoy the option of pouring a bath with the automatic bath fill stop so you can give the resident your full attention while filling the bath. The water level can be freely selected and is set once. You can always continue pouring the bath after the fill stop by pressing the button.



Disinfection system ...

The integrated disinfection offers maximum protection against cross-infections and allows the use of suitable, approved and your own surface disinfectant concentrates. The disinfection shower head and containers are also protected against incorrect operation.

TECHNICAL DATA FOR AVERO VIVA

Total size (incl. operating panel)	2,025 mm x 786 mm
Inside width of upper tub	706 mm
Lifting height	683 - 1,083 mm
Max. patient weight	250 kg
SWL	400 kg
Power connection	230 Volt

ACCESSORIES / OPTIONS

Automatic water stop
Change of viewing direction
Tub shortener
Disinfection
Air-Jet Whirl System
Coloured light therapy system
Music sound system
Aroma therapy system
Power failure emergency function
Special colours according to RAL

MMS Medical,
51 East Gate Drive,
Little Island
Cork.

T: 021 4618000
F: 021 4618099
E: info@mmsmedical.ie

MMS Medical,
Unit 81 Western Parkway
Business park,
Lower Ballymount Road,
Dublin 12

T: 01 4500582
F: 01 4565889
E: sales@mmsmedical.ie

www.mmsmedical.ie

