



# Quick guide to your Harmony Home Lift installation

## Please Note

Our engineers will be responsible for ensuring your safety as well as their own during their time in your home.

This leaflet is for guidance only. If you require further details, please contact us via one of the means listed below.

**MMS Medical**  
51 East Gate Drive, Little Island,  
Co. Cork

Tel : 021 461000  
E: [info@mmsmedical.ie](mailto:info@mmsmedical.ie)  
[www.mmsmedical.ie](http://www.mmsmedical.ie)




**Terry Group Limited**  
1-3 Longridge Trading Estate  
Knutsford, Cheshire, WA16 8PR

Tel: 0345 365 5366  
Web: [www.terrylifts.co.uk](http://www.terrylifts.co.uk)  
Email: [sales@terrylifts.co.uk](mailto:sales@terrylifts.co.uk)




# 10 simple steps to install your home lift

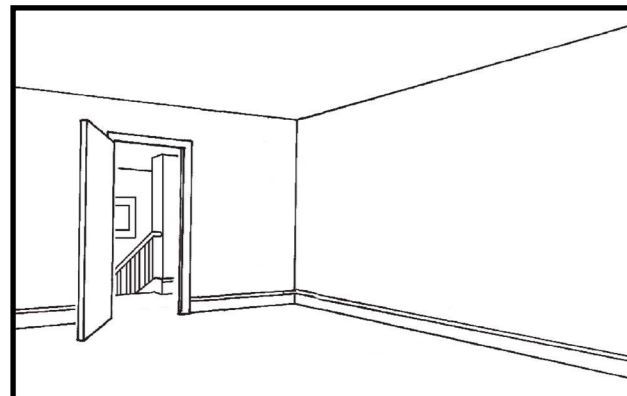

**1**


Upstairs:  
 • Whole room to be cleared by the customer.  
 Downstairs:  
 • Area around the lift installation to be cleared by the customer.

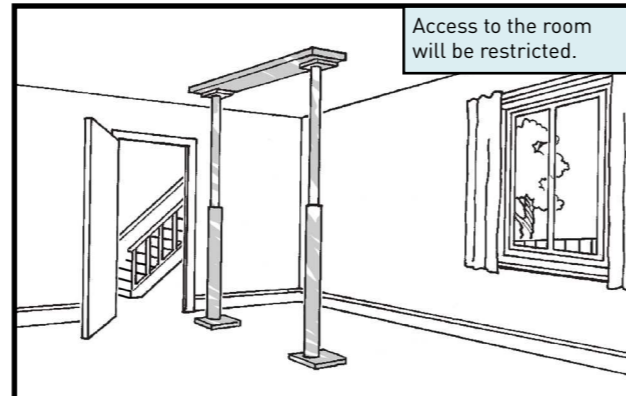

**2**

Downstairs we will:  
 • Support the downstairs ceiling with adjustable props prior to forming the aperture.


Access to the room will be restricted.

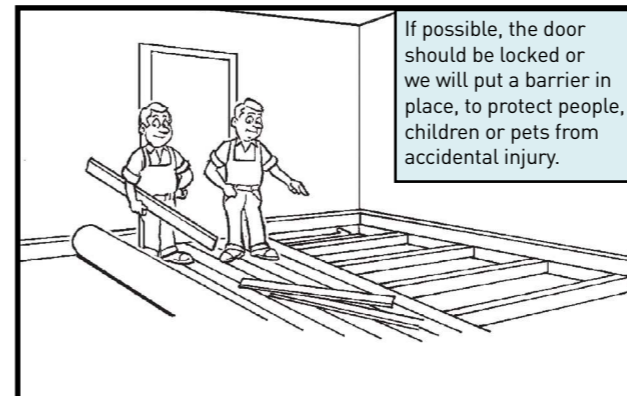

**3**

Upstairs we will:  
 • Roll back the carpet at the upper floor and remove the necessary floorboards to create an aperture.  
 • Please be aware that a certain amount of noise and dust will be created during this process.


If possible, the door should be locked or we will put a barrier in place, to protect people, children or pets from accidental injury.

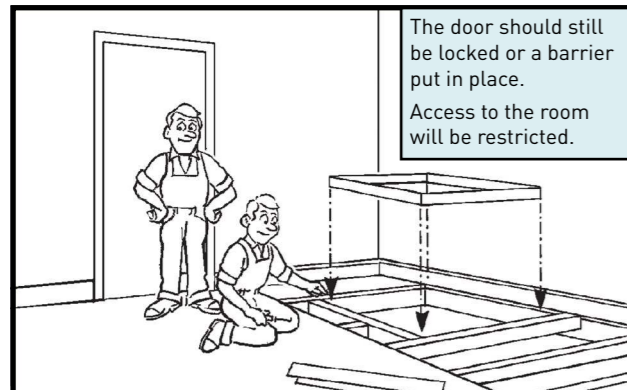

**4**


Upstairs we will:  
 • Replace the floorboards and fit a temporary cover to the aperture.

The door should still be locked or a barrier put in place. Access to the room will be restricted.

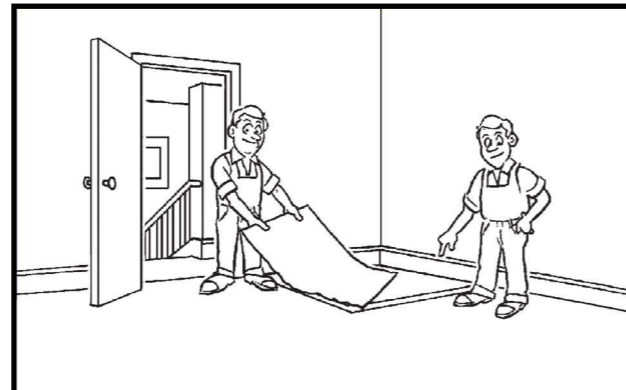


**5**

Upstairs we will:  
 • Roll back the carpet and cut around the aperture. This piece of carpet will be refitted to the top infill panel.

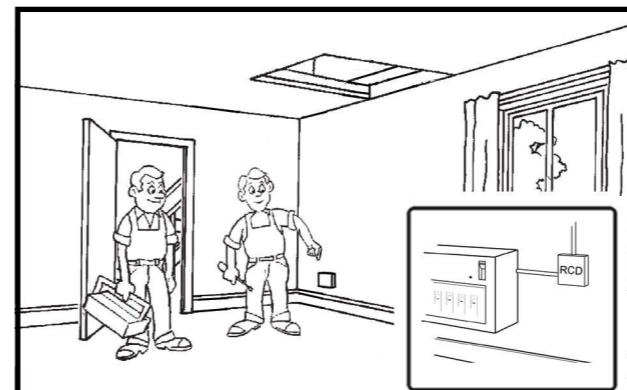


**6**

We will:  
 • Provide a 13A fused spur adjacent to the power pack location. Wiring runs back to consumer unit. A telephone socket may be fitted at this stage if required.

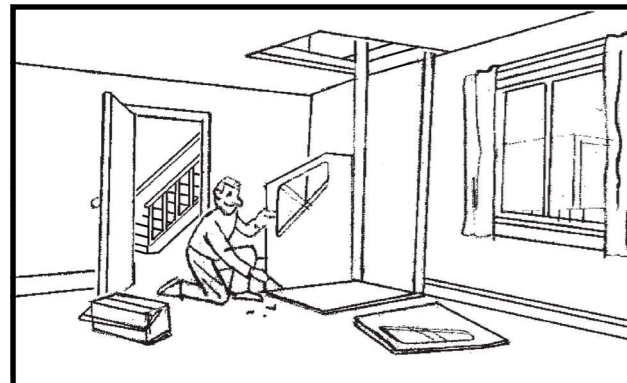


**7**

We will:  
 • Assemble your Harmony Lift.  
 • Clean and tidy the area.

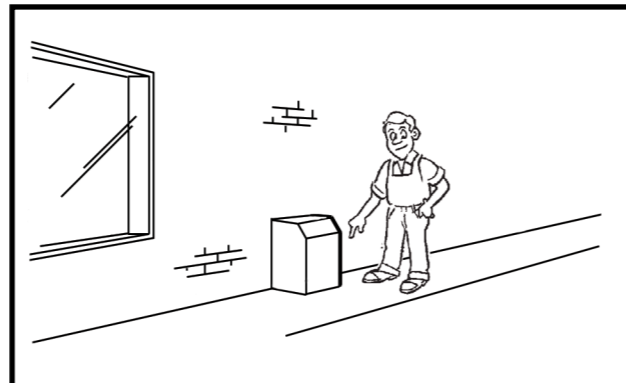

**8**

We will:  
 • Install the power pack internally or externally in the position agreed.

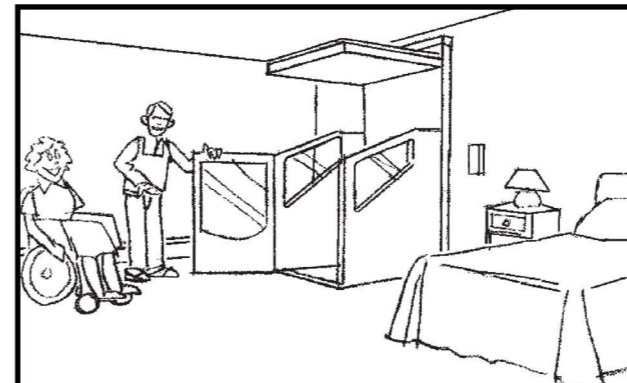

**9**


We will:  
 • Go through the operation of the Harmony Lift with you.  
 • Make final adjustments and ensure you are familiar with all the safety features.

## Easy independent living

- **Market leading range** of lift car options for every need
- **Smart car design** optimises use of space and minimises the lift's footprint
- **Low floor** ensures easy wheelchair access & egress
- **Powered door** on all models as standard
- **Adjustable wireless controls** in car and at both floors
- **Fire and smoke protection** maintained between floors, whether the lift is parked upstairs or downstairs

## Installation timeline

- 3-5 working days' notice will be given to advise you that we are ready to install your lift
- Builders' preparation work will take 1-2 days depending on site conditions
- The installation will take 2 days on average

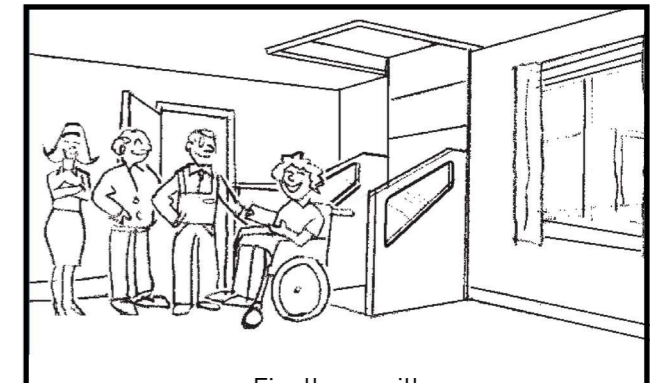

**10**

Finally we will:  
 • Complete the official hand over of your Harmony Lift to you and your family and of course fully demonstrate how the lift should be used along with highlighting the benefits the lift will bring to you all.