


beka 
hospitec

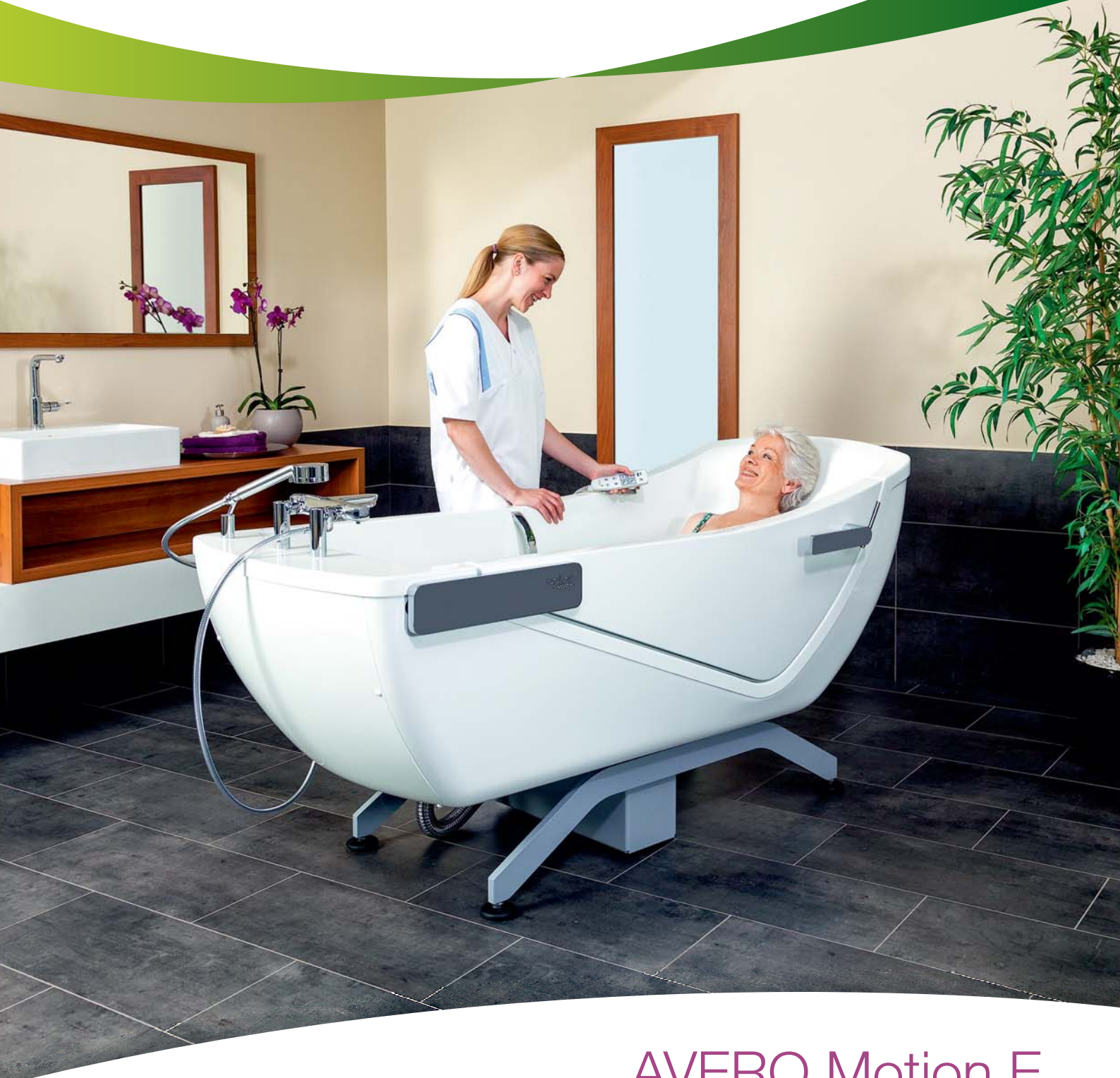

AVERO Motion E

Unique seat and tilting bathtub

Electrical height and reclining adjustment using cable remote control

MMS
MEDICAL

Suitable for all mobility levels

Elegant design and unique function ...



Designed by Uwe Spannagel™



Improves the mobility of your residents ...



The concept of the AVERO Motion E is to simplify showering or bathing, support the mobility of the resident and give him the opportunity to take a relaxing full bath.

With a big door opening to the side, it is possible for mobile and partly mobile patients to enter the bathtub on their own. Only just mobile residents look forward to an independent access to a full bath, without using any tool or without risking to slip on entering the bathtub.

The base frame is freely accessible from underneath, and transfer by means of a NORA active hoist or a CARLO sling hoist is thus possible. Due to the electrical height adjustment of the bathtub, the entry height can be lowered to 49 cm which allows for easy self-entry even from a wheelchair.



Designed by Uwe Spannagel™



Optional change of the viewing direction ...

The unique bathtub design of the AVERO Motion E allows a change of the viewing direction so that your resident can look freely anywhere in the room when bathing or showering without distracting operation panel.

The clear and soft shape of this tub reminds of modern, contemporary private bathrooms and perfectly fits in the existing wellness and bathing concept of your bathroom fittings. The ergonomic tub interior body with its special back profile allows for an optimum sitting and reclining position of the bather.

Electrical height and reclining adjustment, and options such as automatic filling of the tub, coloured light therapy and the air spa system are operated via cable remote control.



Maximum space offered for smaller and bigger residents ...

In developing the AVERO Motion E, care was taken to ensure that there was enough space and comfort even for bigger and heavier residents ...

Maximum space offered

Compared to many other bathtubs, the AVERO Motion E has more space in the seat and shoulder areas precisely for bigger residents. The newly developed inner profile of the tub further offers a large and comfortable lying area for smaller and bigger residents.



The ergonomic inner structure and homely character of the tub shape provide an overall sense of protection to the bather and in this way greatly enhance his well-being.



Comfortable interior with abundant space in the shoulder and seat areas.

Low space requirement

The dimensions of the AVERO Motion E are significantly smaller than those of a conventional care tub. Savings of up to 50 cm of space can be made compared to a standard height-adjustable tub, but without reduction of the inner dimensions of the tub for the bather. You can position the tub in the room wherever you like. A change of the viewing direction for a free view of the resident into the room is easy to arrange with the AVERO Motion E.

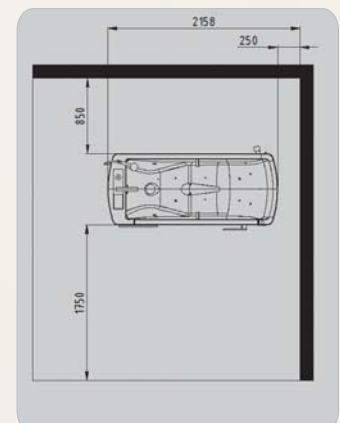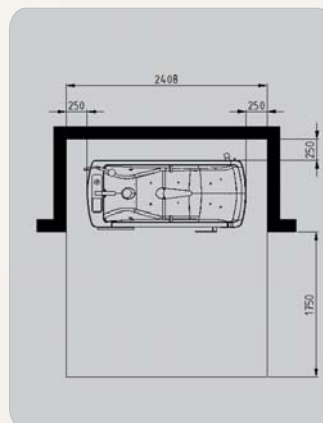

Space requirement for niche position Space requirement for island position

The optimum solution for your nursing staff and your residents ...

The AVERO Motion E seat and tilting bathtub offers both your resident and your nursing staff a maximum of safety and comfort at all times ...

Benefits for the resident

- ❑ Ergonomic foot area for optimum support
- ❑ Adjusted sitting and lying profile for more safety to prevent slipping
- ❑ Long inner profile for big residents also
- ❑ Change of the viewing direction into the room possible, without distracting operation panel
- ❑ Ergo-handles = optimum support for the bather
- ❑ Comfortable inner width for heavy residents
- ❑ Overflow of up to 5 cm higher than other bathtubs in the market = bathing water up to the chest
- ❑ No edges in the door area = simple transfer with wheelchair
- ❑ Seat area with hygiene opening fitting for better hygiene of the intimate areas
- ❑ Large priming volume of 98 l = The resident can lie in warm water immediately after the tub is reclined



Benefits for the nursing staff

- ❑ Automatic pre-filling (optional)
- ❑ Quick draining of the tub water at 75 l per minute
- ❑ Electric height adjustment (up to 20 cm) for optimum working height
- ❑ Hand shower always ready to hand on the tub rim, no water on the floor
- ❑ Cable remote control with all functions: On/off, reclining function, water, hand shower, air spa system and light therapy
- ❑ Integrated disinfection with disable function (optional)
- ❑ Scalding protection for tub spout and hand shower
- ❑ Door latch with additional safety lock for max. safety




Options AVERO Motion E

Wellness and therapy ...



A full bath has beneficial effects on the body and the mind. According to the latest findings in dementia research, sensory impulses, in particular, have shown a therapeutic effect on demented residents ...

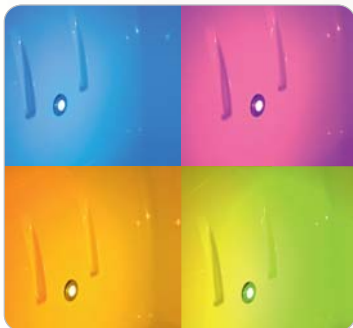

Coloured light therapy system ...

The coloured light therapy system of the AVERO Motion E creates a special and optically stimulating atmosphere of well-being for your resident. By means of LED lights integrated in the foot area, the bathing water can be immersed into the entire colour spectrum without heat radiation and without blinding the resident unnecessarily. The coloured light therapy system has a freely selectable colour option and an automatic sequence programme for colour sequences.



Air spa system ...

An integrated air spa system with its cleansing and vitalising abilities provides a comfortable massaging effect which is highly beneficial to the resident. The slight tingling on the skin is relaxing and soothing, especially in case of restless patients, and further promotes the blood circulation of the skin. Contamination of the integrated jets in the tub bottom and the foot area is not possible due to a special process, and there is therefore no need for an integrated disinfection system to disinfect the air spa system.



Music sound system ...

With two sound pressure generators integrated in the area of the upper body part, the vibrations of the music are transmitted directly into the water and to the resident's body. This creates a tactile sound experience which has a positive influence on the bather, as shown by the latest research. A USB port allows to play the music chosen by your resident. It is operated via wireless remote control.

When it comes to colour, you can, of course, select from a variety of RAL colours for our AVERO Motion E seat and tilting bathtub ...



Aromatherapy system ...

An aromatherapy system integrated in the tub body allows for different aromas to create a wonderful experience for the senses of the resident in conjunction with the air heated by the air spa system and the bath water. It goes without saying that the operation is simple and the system is maintenance-free. This option is available with the air spa system only.



Automatic filling of the tub ...

Automatic filling of the tub allows for independent pre-filling of the foot area with up to 98 litres of water. This offers the advantage to the resident, on the one hand, that he can lie down in warm water immediately after the door is closed and the tub is reclined, and, on the other hand, this option also means saving a significant amount of time for the nursing staff which will be beneficial to the resident.



Disinfection facility ...

The disinfection facility installed offers maximum protection against cross-infections and allows the use of suitable, authorised, in-house surface disinfectant concentrates. The disinfection container is secured with a lockable cover and situated in the leg frame of the bathtub and can be easily replaced.

Technical data of the AVERO Motion E

Tub length (entry position/ reclined position)	1.760 mm / 1.908 mm
Tub width (outside)	860 mm
Lowest entry height	490 mm
Working height	810/1.010 mm
Inclination angle	23°
Draining of tub (Litre/min.)	75 l/min
Power supply	230 Volt, 50 Hz, max. 5 Ampere
Patient weight (max.)	210 kg
SWL	300 kg
Pre-fill volume (foot area)	98 l

Accessories / Options

Change of viewing direction
Automatic water stop
Disinfection facility
Air spa system
Coloured light therapy system
Music sound system
Aromatherapy system
Comfort neck cushion
Second holder for hand shower
Special colours as per RAL



MMS Medical Ltd.
51 East Gate Drive
Little Island,
Cork.

Tel: 021 4618000
Fax: 021 4618099
E: info@mmsmedical.ie

MMS Medical Ltd.
Unit 81, Western Parkway Business Park,
Lower Ballymount Road,
Dublin 12.

Tel: 01 4500582
Fax: 01 4565889
www.mmsmedical.ie