

beka 
hospitec



SENTA PUR L

The combination lift for bathing and showering

MMS
MEDICAL

The SENTA PUR L combination lift

The ideal solution for bathing and showering ...



Fits in with every modern care tub ...



The SENTA PUR L showering and bathing lift is exactly the right tool to be able to shower or bathe the person needing help safely and comfortably at any time, regardless of the level of mobility of your resident. The SENTA PUR L lift is extremely easy to manoeuvre and can be applied in any type of care tub in a flexible manner.

Height adjustment is stepless via a battery-operated electric drive and therefore not distressing for the resident. The lift is operated easily and safely via a corresponding hand-held remote control and can be handled by a single staff.

Each lift is equipped with standard protection devices against overload and against collision. An integrated service module reports disturbances and maintenance due to the lifter. An integrated manual emergency lowering device is, of course, included.



Easily convertible into a stretcher lift ...

The SENTA PUR L bathing lift can be easily and quickly converted into a fully adequate stretcher lift by means of optional stretcher lift modules. By default, each seat is prepared in such a way that one single staff is able to plug in the side wings without effort. This provides you with the necessary flexibility that is required without having to resort to a second lift. The seating area has a large care opening which can be closed with a cover if required.

Flexibly applicable - also for transfer from the bed. Depending on your care routine, you can also pick up your resident from the bed with our showering and bathing lift directly. The deep carrier frame allows to be passed underneath the care bed for easy transfer from the bed. Your resident can, of course, also sit down on the seat by himself. The carrier frame is formed in a way that this is possible without hindering the legs and heels.



SENTA PUR L combination lift

Offers the highest quality and comfort ...



Offers a variety of options ...



Integrated calibrated patient scale. An important factor in current care documentation is ascertaining and monitoring the weight of the resident. Our SENTA PUR L lift with integrated patient scale offers an ideal option to weigh the resident without another transfer becoming necessary.

Stretcher lift modules with optional foot rest. Two removable stretcher lift modules can easily convert the seat lift into a full stretcher lift. The modules are simply clicked into an existing safety adapter at the seat tray by means of a safety click system. Additionally, you can also obtain a pluggable foot rest.

Side rails. An optional side rail gives your resident additional hold and safety. The bracket is located under the seat and can be slid back under the seat if the side rails are not used.



Benefits for your nursing staff and your residents ...

- ❑ Easy handling by a single staff
- ❑ Electric height adjustment with emergency lowering device
- ❑ Versatile system, applicable as seat or stretcher lifter
- ❑ Fits in with any modern care tub
- ❑ High level of safety by means of holding fixtures
- ❑ Skin-friendly padding made of PUR material
- ❑ Safe transfer to and from the tub
- ❑ Easy self-entry
- ❑ Service and disturbance display
- ❑ Materials made of stainless steel and aluminium
- ❑ Secured, easy-to-move double castors
- ❑ Manual keypad with visual display of the battery charge status
- ❑ Battery and charger compatible with all mobile BEKA Hospitec hoists



Characteristics of the SENTA PUR L

Your advantages for safe bathing and showering ...

Fits in with all modern care tubs, with electric height adjustment, large hygiene opening, convertible into a stretcher lift, removable battery, specially designed carrier frame, easy-to-move double castors, manual emergency lowering system, ergonomic handles, and many options ...



1 The carrier frame made of high-quality steel is formed in a way that it allows for self-entry.



2 Hand-held remote control with disturbance and service display and permanent display of the battery charge status.



3 The battery can be removed at the side for narrow doors.



4 Closed, easy-to-move double castors, 2 of which have brakes.





5 **Optional:** Digital patient scale, calibrated.



6 **Optional:** Digital patient scale, not calibrated.



7 Seat area with large, lockable hygiene opening.



7




8 **Optional:** Side rail for plugging into a bracket at the seat with safety lock.



9 **Optional:** Bedpan support for inserting your in-house bedpan.



4

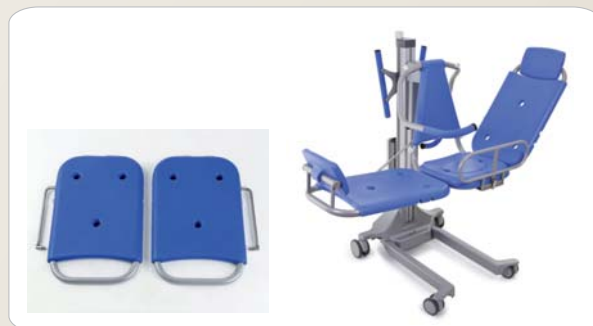

10 **Optional:** Two removable stretcher lift wings comfortably convert the seat lift into a full stretcher lift. The two modules are easily clicked into an existing safety adapter at the seat tray. Optionally also available with a foot rest.



11 The PUR padding is easily slid onto the stainless steel frame and can be removed at any time.

Technical data of the SENTA PUR L

Max. patient weight	150 kg
Lifting range	475 mm - 1.175 mm
Lifting power	700 mm
Length	860 mm
Width	718 mm
Height	1.375 mm
Two double castors (front)	diameter 100 mm
Two double castors (rear, with locking device)	diameter 125 mm
Battery (removable)	24 V
Circuit break switch and emergency lowering system	

Accessories/Options

Digital patient scale
Stretcher lift modules (2 pieces)
Additional grab handle for the bathing lift
Side rail
Foot rest
Bedpan support
Replacement battery
External wall charging unit



MMS Medical Ltd.
51 East Gate Drive
Little Island,
Cork.

Tel: 021 4618000
Fax: 021 4618099
E: info@mmsmedical.ie

MMS Medical Ltd.
Unit 81, Western Parkway Business Park,
Lower Ballymount Road,
Dublin 12.

Tel: 01 4500582
Fax: 01 4565889
www.mmsmedical.ie